



Ivory® Rubber Dams and Clamps

The complete system for tooth isolation.

Increasing practitioner efficiency and patient safety, rubber dams are the cornerstone of excellent restorative and modern dentistry. The Ivory Rubber Dam system by Kulzer is designed to help you perfectly isolate the treatment area so that you can do your best work free from distraction.

Giving a hand to oral health.



KULZER
 MITSUI CHEMICALS GROUP



Rubber dam – isolate better, work better.

Flexible options for uncompromising isolation.

Rubber dams are proven to increase the success of endodontic, restorative and cosmetic treatments because it provides the clean and dry working field you need to place adhesive restorations that last.

Made from natural latex that has been washed in the manufacturing stage to reduce surface proteins, the original Ivory Rubber Dam* material is thin and pliable yet boasts an extremely high tear strength.

Additionally, the Ivory Rubber Dam line gives you flexible freedom of choice in selecting the rubber dam material that's perfect for your needs. You can choose latex-free material, black-colored material for very high visual contrast, and even mint-flavored material to give your patients a more pleasant experience.

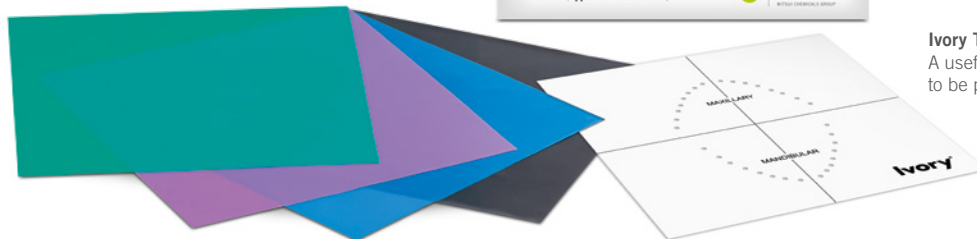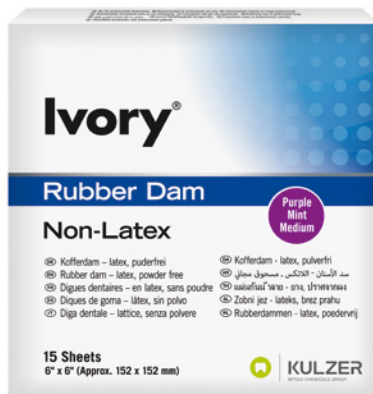

Ivory Template
A useful guide for easily marking holes to be punched out of the rubber dam.

- Powder-free, thin, comfortable sheets
- 100 % natural latex with no added fillers
- Latex-free material also available
- Pleasant mint flavor
- High-contrast black-colored material for better visibility
- Extremely high tensile strength**
- Expansion up to 10x from the original punch hole**
- Prevents irritation and damage of soft tissues

* The legal production site for Ivory Rubber Dam is Sanctuary Health Sdn Bhd, No 16, Persiaran Perindustrian Kanthan 1, Kawasan Perindustrian Kanthan, 310200 Chemor, Perak, Malaysia.

** Internal. Data on file.




Ivory accessories

Durable and reliable.

Ivory® Punch


This precision instrument is made from high-quality stainless steel and has a rotating cutting disc and a unique floating plunger that strikes the cutting disc evenly, guaranteeing a tear-resistant and perfectly sized hole for any tooth.

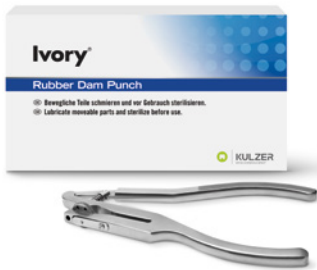

Ivory® Forceps

Place rubber dam clamps with ease.

You'll get many years of reliable use out of these Ivory stainless steel forceps. These forceps will reach around the bow of any Ivory clamp without tilting, which is especially important when reversing a clamp. The straight head design makes it easy to grip and place Ivory clamps of any size.

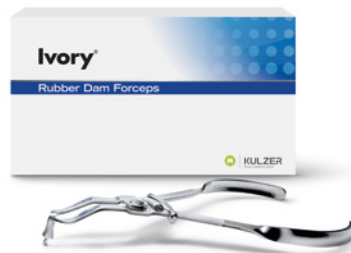

With Ivory® Rubber Dam materials and instruments by Kulzer, you can make the use of rubber dam a fundamental part of your treatment approach and start enjoying all the benefits.

Ivory® Rubber Dam Kits

Ivory® Rubber Dam Kits supply you with everything you need to use rubber dams with confidence. All instruments are designed for durability and ease of use and are made from high-grade stainless steel.

Not sure which clamps you need?

The **Ivory® Rubber Dam Clamps Starter Kit** offers a varied selection of popular general-use clamps that cover most needs.



This accessory is made of stainless steel and auto-clavable. It is available in two sizes – children 5" x 5", adults 6" x 6".

You want to have everything at hand?

The **Ivory® Rubber Dam Complete Kit** is a self-contained unit that includes the instruments you need to get started.

Complete Kit


- 1x Box 6" x 6" Blue Medium Rubber Dam
- 1x 6" x 6" Marking Template
- 1x 6" Frame
- 8x Most Popular Clamps (0, 2A, 8A, 14, 2, 7, 9, 14A)
- 1x Clamp Forceps
- 1x Rubber Dam Punch



Ivory® Rubber Dam Clamps


A secure hold for absolute isolation.

Choose from a wide variety of uniquely designed Ivory Rubber Dam Clamps to find the perfect one for each tooth. Our line includes a clamp for every application. Every single clamp is crafted in the USA from high-grade stainless steel and carefully hand-set so that you can confidently affix the rubber dam whenever it's needed.

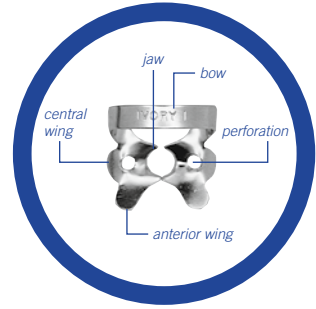

Molar

3



For small lower molars. Narrow, flat jaws so as not to impinge in tissue.

3
W3

4



For small uppers. Narrow jaws adapt to fit round cervical (neck) collars.

4
W4


5



For large upper molars. Jaws adapt to curvature at the gum line.

5
W5


7



General purpose lower molar. Medium-sized bow and jaws. Flat jaws. For teeth which tend to be square in form.

7
W7


7A



For large lower molars. Same as No. 7, only larger. Large bow, broad jaws.

7A


8



For small upper right and lower left molars with short crowns. Buccal jaw broader than lingual.

8
W8

8A



For upper, lower molar roots. Four-point jaws grasp root at corners. Jaws set dam beneath gum line when released.

8A
W8A


10



For small upper left and lower right molars with short crowns. Buccal jaw broader than lingual.

10


11



For small upper right and lower left molars with short crowns. Buccal jaw broader than lingual.

11


12A



For all lower right molars, particularly third molars and partially erupted teeth. Available with serrated jaws only.

12A

13A



For all lower left molars, particularly third molars and partially erupted teeth. Available with serrated jaws only.

13A

14



For upper and lower molars, partially erupted, round teeth or for teeth sloping from the gum line. Small bow.

14
14DT
14T
W14

14A



Same as No. 14, only larger in every way.

14A
W14A

26



For small lower molars.

26

26N



Universal upper with festooned jaws.

26N

27



For large upper and lower molar distal cavities.

27

28N



General purpose lower molar clamp.

28N

56



For all large upper and lower molars.

56
56T
W56

Central Anteriors

00



For small lower anteriors and teeth set irregularly in arch. High bow, narrow jaws.

00

6



For anterior labial and premolar teeth.

6


9



For anterior labial teeth.

9
9T

Tooth Diagram

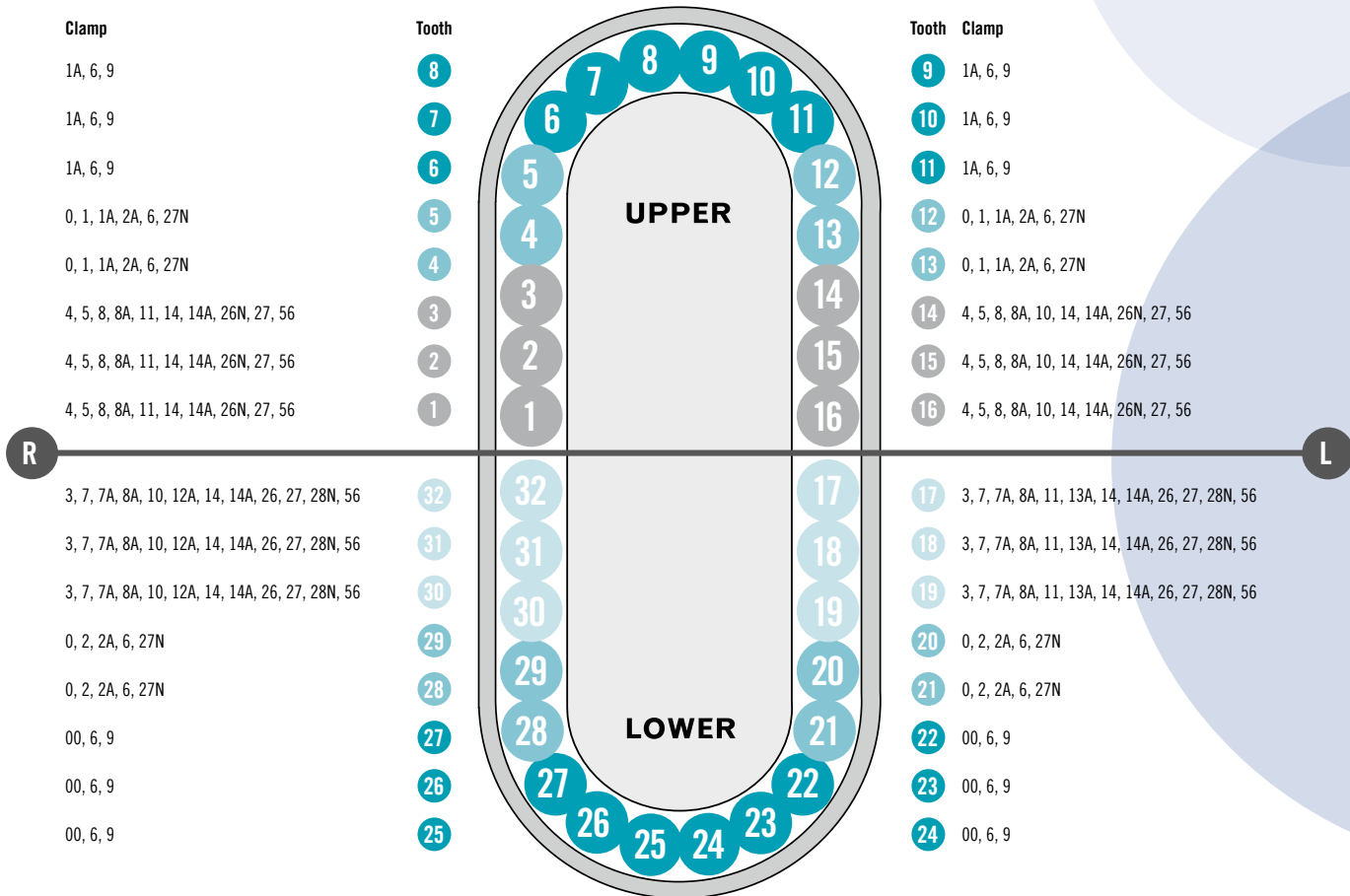

Premolar

<p>0</p> <p>For upper and lower premolar teeth that are elongated or have small necks.</p> <p>0 W0</p>	<p>1</p> <p>General purpose upper premolar. Jaws adapt to curvature at the gum line.</p> <p>1 1T W1</p>	<p>1A</p> <p>Upper premolar and anterior root clamp. Jaws adapt in a downward slope to grasp roots.</p> <p>1A</p>
<p>2</p> <p>General purpose lower premolar. Same as No. 1, only jaws are flat so as not to impinge on tissue.</p> <p>2 2T W2</p>	<p>2A</p> <p>For upper and lower premolars with large necks. Broad jaws.</p> <p>2A 2AT W2A</p>	<p>27N</p> <p>Universal premolar clamp. Jaws adapt to curvature at the gum line.</p> <p>27N</p>

DT Distal access, serrated jaws
T Also available with serrated jaws (Tiger clamps)
A Slightly larger
W/N Wingless

Which Ivory® Rubber Dam Clamp do you need?

Listed below is the complete Ivory Rubber Dam Clamp line, along with guidelines to help you select the right clamp for the job.



Most Popular Clamps

PREMOLARS – WINGED

- 00 Small lower, anterior
- 0 Elongated small neck
- 1 General purpose upper
- 2 General purpose lower
- 2A Large neck premolar

PREMOLARS – WINGLESS

- W2 General purpose lower
- W2A Large neck premolar

MOLAR – WINGED

- 3 Small lower
- 4 Small upper for round teeth
- 7 General purpose lower
- 8 General purpose upper
- 8A Upper and lower molar root
- 12A Right partially erupted, 3rd molar
- 13A Left partially erupted, 3rd molar
- 14 Partially erupted, most popular clamp

MOLAR – WINGED (CONTINUED)

- 14A Large, partially erupted
- 56 All large molars

MOLAR – WINGLESS

- W3 Small lowers
- W8A Upper and lower molar root
- W14A Large partially erupted

Specialty Clamps

PREMOLARS – WINGED

- 1A Root clamp

PREMOLARS – WINGLESS

- W0 Elongated small neck
- W1 General purpose upper
- 27N Universal

MOLAR – WINGED

- 5 Large upper
- 7A Large lower
- 10 Right, small short crown

MOLAR – WINGED (CONTINUED)

- 11 Left, small short crown
- 26 Small lower molar
- 27 Distal access, large molar

MOLAR – WINGLESS

- W4 Small upper
- W5 Large upper
- W7 General purpose lower, square teeth
- W8 General purpose upper, round teeth
- W14 Partially erupted

MOLAR – WINGLESS (CONTINUED)

- W56 Large molar
- 26N Universal upper
- 28N General purpose lower molar

LABIAL – WINGED

- 6 Central and incisor
- 9 Labial prep, all anteriors

Tiger Clamps

- | | |
|------------------------------------|-------------------------------------|
| 1T Premolar, general purpose upper | 9T Anterior labial |
| 2T Premolar, general purpose lower | 14T Molar, partially erupted |
| 2AT Premolar, large neck premolar | 14DT Molar, distal access, serrated |
| | 56T For large molars |

Accessories

- | | |
|---|------------------------------------|
| Ivory Rubber Dams – green, blue, black and purple, 5" x 5" or 6" x 6" | Ivory Rubber Dam Punch |
| Ivory Rubber Dam Template | Ivory Rubber Dam Clamp Starter Kit |
| Ivory Rubber Dam Frames | Ivory Rubber Dam Complete Kit |
| Ivory Rubber Dam Forceps | |

Ivory Stainless Steel Clamps

Premolar Winged Clamps







50057302	Stainless Steel Clamp No. 00
50057304	Stainless Steel Clamp No. 0
50057306	Stainless Steel Clamp No. 1
50057310	Stainless Steel Clamp No. 2
50057312	Stainless Steel Clamp No. 2A
50057322	Stainless Steel Clamp No. 3
50057324	Stainless Steel Clamp No. 4
50057328	Stainless Steel Clamp No. 7
50057334	Stainless Steel Clamp No. 8
50057336	Stainless Steel Clamp No. 8A
50057348	Stainless Steel Clamp No. 12A
50057352	Stainless Steel Clamp No. 13A
50057354	Stainless Steel Clamp No. 14
50057356	Stainless Steel Clamp No. 14A
50057374	Stainless Steel Clamp No. 56

Premolar Wingless Clamps

50057510	Stainless Steel Clamp No. W2
50057512	Stainless Steel Clamp No. W2A
50057522	Stainless Steel Clamp No. W3
50057536	Stainless Steel Clamp No. W8A
50057556	Stainless Steel Clamp No. W14A

Rubber Dam Products

Rubber Dam for adults (6" x 6")

66094052	 Blue Thin	36 Sheets
66094053	 Blue Medium	36 Sheets
66094054	 Blue Heavy	36 Sheets
66094055	 Green Medium (Mint)	36 Sheets
66094056	 Black Medium (Mint)	36 Sheets
66094057	 Purple Medium (Non Latex, Mint)	15 Sheets

Rubber Dam for children (5" x 5")

66094058	 Blue Medium	52 Sheets
66094059	 Green Medium (Mint)	52 Sheets
66094060	 Purple Medium (Non Latex, Mint)	15 Sheets

Rubber Dam Kits

50057968	Rubber Dam Clamps Starter Kit: contains 8 Clamps: 0, 2, 2A, 7, 8A, 9, 14, and 14A
50057966	Rubber Dam Complete Kit: contains Punch, Forceps, 6" x 6" Rubber Dam, 6" Frame, 8 Clamps and Template

Rubber Dam Pliers

50057220	Rubber Dam Forceps
50057225	Rubber Dam Punch

Rubber Dam Frames

50057230	Adult Frame (6")
50057229	Pedo. Frame (5")

Rubber Dam Template

66095030	Template 6" x 6"
----------	------------------

Ivory Stainless Steel Clamps

Winged Clamps

50057308	Stainless Steel Clamp No. 1A
50057326	Stainless Steel Clamp No. 5
50057330	Stainless Steel Clamp No. 7A
50057338	Stainless Steel Clamp No. 10
50057342	Stainless Steel Clamp No. 11
50057370	Stainless Steel Clamp No. 26
50057372	Stainless Steel Clamp No. 27

Wingless Clamps

50057504	Stainless Steel Clamp No. W0
50057506	Stainless Steel Clamp No. W1
50057524	Stainless Steel Clamp No. W4
50057526	Stainless Steel Clamp No. W5
50057528	Stainless Steel Clamp No. W7
50057534	Stainless Steel Clamp No. W8
50057554	Stainless Steel Clamp No. W14
50057728	Stainless Steel Clamp No. 26N
50057730	Stainless Steel Clamp No. 27N
50057732	Stainless Steel Clamp No. 28N
50057568	Stainless Steel Clamp No. W56

Ivory Anterior Clamps

50057382	Stainless Steel Clamp No. 6
50057384	Stainless Steel Clamp No. 9

Ivory Tiger Clamps

50057841	Stainless Steel Clamp No. 1T
50057843	Stainless Steel Clamp No. 2T
50057847	Stainless Steel Clamp No. 2AT
50057849	Stainless Steel Clamp No. 14T
50057857	Stainless Steel Clamp No. 14DT
50057853	Stainless Steel Clamp No. 56T
50057855	Stainless Steel Clamp No. 9T



GLUMA® Desensitizer – More gain, less pain.

For more than 20 years, one drop of GLUMA Desensitizer has been all you need to stop and prevent hypersensitivity. The result is fast and effective, without mixing, curing or repetitive steps. And your patients feel a prompt relief. Use GLUMA Desensitizer under every restoration – direct and indirect – to ensure your patients the comfort they deserve. Your patients appreciate the fast, effective and long-term results.

Prevent glutaraldehyde-induced irritation of the gingiva:
Use Ivory® Premium Rubber Dam

GLUMA Desensitizer's benefits at a glance.

- | | |
|-------------------------------|---|
| One-step placement | ▶ No agitation or light curing required makes it easy to use and saves time |
| Universally compatible | ▶ GLUMA Desensitizer can be used with bonding and restorative materials, in any situation |
| Economical | ▶ One drop, multiple applications |



Contact in the USA

Kulzer, LLC
4315 S. Lafayette Blvd.
South Bend, IN 46614
Phone: (800) 431-1785
Fax: (800) 522-1545
customer.servicehkn@kulzer-dental.com

Contact in Germany

Kulzer GmbH
Leipziger Straße 2
63450 Hanau, Germany
info.dent@kulzer-dental.com