




Shipping cara® Print 4.0 Demo Units For Service or Repair Packaging Guide

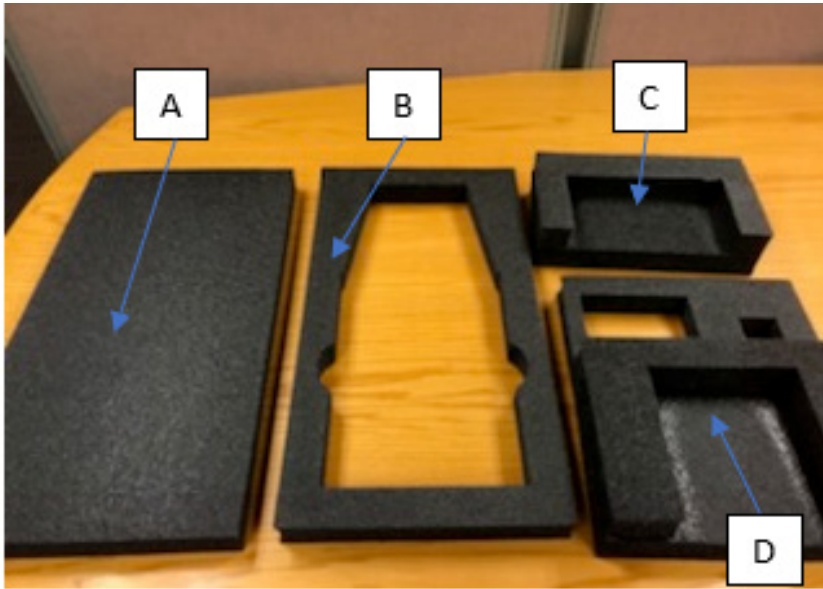


Shipping Foam Pieces: A, B, C and D

1. Seal printer in plastic bag (E) with desiccant (F).



2. Place foam (A) and (B) in shipping box. Foam (A) goes in first, foam (B) is placed on top of foam (A).

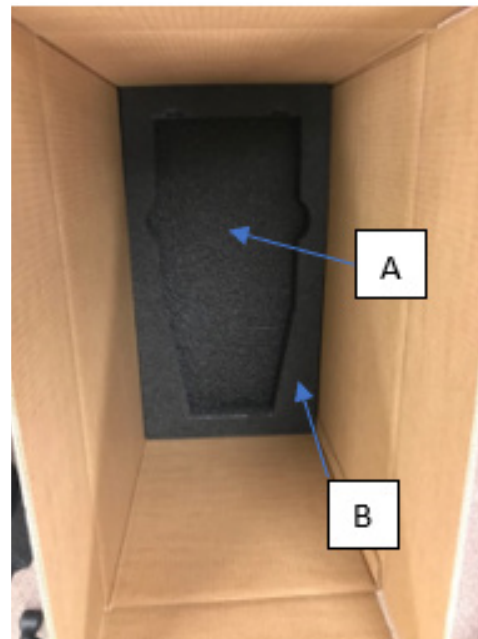

3. Place sealed printer in shipping box. Confirm orientation of printer contour conforms with cut out in foam pieces (B).



Questions?

Kulzer Service for cara Print 4.0 can be reached by emailing Cara-Service-NA@kulzer-dental.com or calling our toll-free number and asking to speak with a technical service consultant: **(800) 431-1785**

4. Place foam (C) and (D) as shown in box, nesting with printer. Add any additional foam padding you may have to protect the top and sides of the printer. NOTE: Please **DO NOT** include the build table, power supply cord, or any accessories. **Ship the printer only!**



Foam at head of printer



Foam at foot of printer