


CARA PRINT CUBE FIRMWARE

RELEASE NOTES

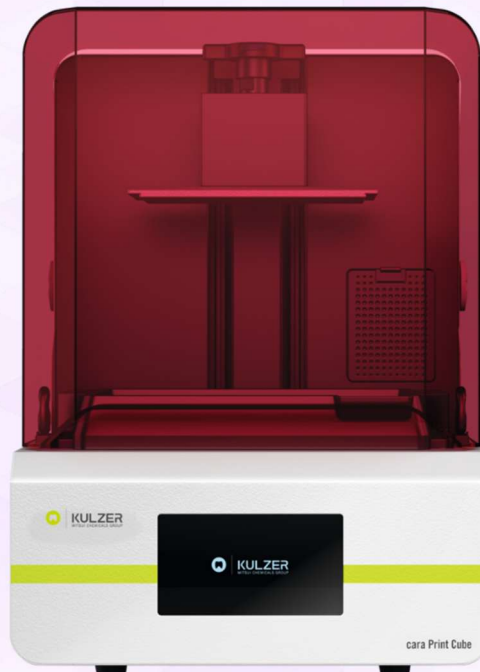

Version: 02
Date: 2026-04-27



[www.kulzer.com/
mycube-support](http://www.kulzer.com/mycube-support)



KULZER
MITSUI CHEMICALS GROUP

General Information

Please always make sure to never rename the “.up” installation file for the cara Print Cube Firmware. If renamed, the printer will not detect the file correctly and the firmware update cannot be updated.

Please also make sure to not hide the “.up” installation file in a folder on your USB drive. It must be saved on the first level when updating. See the training video “Updating the firmware” for further information.

Below, please see a historic list of cara Print Cube Firmware versions including the release date.

cara Print Cube Firmware version	Release date
V1.2.14	2024-11-15
V1.2.15	2025-06-02
V1.2.16*	2026-04-27

*latest version

Revision History

cara Print Cube Firmware V1.2.15	
1	LCD panel actual usage time and lifetime monitoring added
2	Intensity correction for every new LCD panel added. This information is mandatory to insert in combination with the serial number of the LCD panel when installing a new LCD panel.
3	Name of used print parameters now visible in two lines in the print job preview to increase readability.
4	Japanese language added
5	Other language translations improved

cara Print Cube Firmware V1.2.16*

1	Maximum light exposure time limitation in the “Clean Print” function added (10 seconds)
2	Block time added before next “Clean Print” can be started (30 seconds)
3	Simplified display of print job information in print job list
4	Build platform heating: Reset to “auto” after every successful or aborted print job
5	Auto restart of the printer after language change
6	Improved network connection stability (LAN and WiFi)